



Folding Pedal Exerciser

For Safety:

REF 10-0718

- Consult your doctor before starting any exercise program.
- Keep children away from Pedal Exerciser while in use.
- Ensure that the Pedal Exerciser is positioned on a solid flat surface.
- Do Not wear loose clothing that could become caught or trapped during exercise.
- Please exercise at your own pace and Do Not over-exercise which may cause injury.
- Do Not exercise in the standing position.
- Capacity: 150 lbs.

Maintenance:

- Regularly check to see all the nuts, bolts and fittings are securely tightened. Periodically, check all moving parts for signs of wear or damage.
- Clean with a dry cloth. Do Not use solvent cleaners.

Usage:



Unfold to use:
Pull each leg all the way open until the push button pops to lock in place.



Fold to store:
Push and hold the push button on both legs and bend each leg to fold for storage.



Twist to change resistance:
Use the tension control knob to increase or decrease the pedal resistance. Twist knob clockwise to increase resistance or twist knob counterclockwise to decrease resistance.



Fabrication Enterprises Inc.
250 Clearbrook Rd, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



AJW Technology Consulting GmbH
Königsallee 106
40215 Düsseldorf (Germany)