

Standing Platform

PARTS: Standing Platform



ATTACHMENT INSTRUCTIONS

TO BEGIN INSTALLATION: Remove the folding platform and telescoping squat stand. **STEP 1.** Position the Standing Platform mounting brackets over the holes on the lower rail just above the base with the curve of the handles pointing toward the tower. **STEP 2.** Pull the retractable knob on one side of the Standing

Platform and slide the bracket over the hole on the rail as shown. **STEP 3.** Release the retractable knob to secure the pin in place. Repeat on the other side. **STEP 4.** Push and pull the Standing Platform to insure both the pins are engaged completely in the rails.



ACCESSORY IN USE

