

BAPS Adapter

NOTE: The BAPS Adapter is designed for physical therapy and rehabilitation use with the Closed Chain Platform (sold separately) and a BAPS board (not included).





detent pin


ATTACHMENT INSTRUCTIONS

TO BEGIN INSTALLATION: First attach the Closed Chain Platform. Face the front of the tower. **STEP 1.** Align the bottom of the folding platform with the lower rails just above the base. **STEP 2.** Slide the left pins of the folding platform into the holes on the outside of the left rail near the base. **STEP 3.** Pull the retractable pin on the right side of the folding platform and move the pin into the hole on the outside

of the right rail. Release the pin. Adjust the folding platform until the pin engages completely. **STEP 4.** Align the Closed Chain Platform (CCP) over the folding platform struts as shown. **STEP 5.** Pull the CCP Adjust Lever and slide the CCP down the folding platform struts almost as far as it will slide. Release the lever and continue sliding the CCP down until the lever completely engages the holes in the struts.

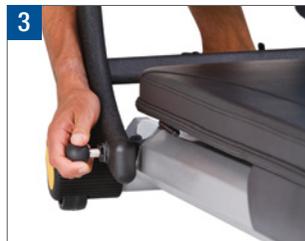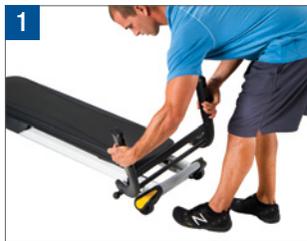

ACCESSORY IN USE

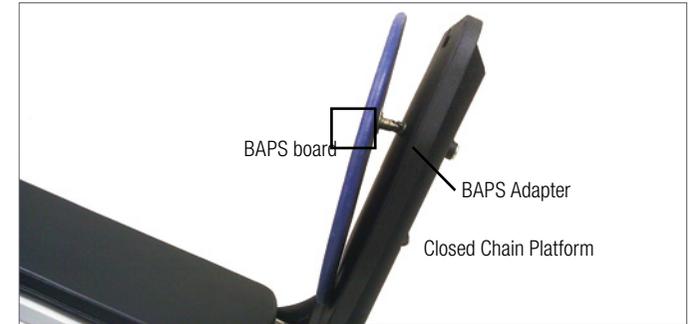

BAPS board

BAPS Adapter

Closed Chain Platform



BAPS board in use with a BAPS Adapter and the Closed Chain Platform.

You may need to lift it slightly for both sides of the lever to pop into the holes. **STEP 6.** To use the BAPS Adapter, insert the base of the BAPS Adapter into the round opening on the front of the CCP. Secure it at the desired height with the detent pin on the back of the CCP as shown.