

INSTRUCTIONS



MOIST HEAT PACKS AND COVERS



Introduction

- Moist heat pack is a simple and effective method of applying localized heat
 - Gives at least 30 minutes of therapeutic moist heat
 - Apply pack with Relief Pak® cover or heavy towel
 - When treatment is complete pack can be stored in a plastic bag in freezer
 - Terry hot pack cover allows therapeutic moist heat to penetrate without burning the skin
 - Covers are filled with foam and additional terry
 - Hook and loop fasteners keep moist heat packs in place in cover
 - Covers are washable
-

Heat your moist heat pack

1. Hold pack by corner loops and gently shake. This distributes the filler and permits it to swell (absorb water) evenly.
 2. Place pack in heating unit (for home, use a large pot or pan).
 3. Fill heating unit with enough water to keep pack completely immersed.
 4. Let pack soak for at least 2 hours. Pack will become completely swollen with water. This long soaking procedure is necessary only with a new pack. After initial use, pack can be prepared by soaking for 15-20 minutes.
 5. If microwaving, place hot pack in microwave safe container and cover completely with water. Place container in microwave and heat at full power until water temperature reaches 160°F (-71°C).
 6. If using stove place pack in heating vessel and slowly increase water temperature to 160°F (-71°C). Let pack soak at 160°F (-71°C) for 20 minutes.
-

Use with Moist Heat Pack Cover

Two basic cover designs are available. The pocketed cover is designed for simplicity, the flat cover is designed for versatility.

Warning: If cover is too hot, use additional toweling.

Pocketed cover (oversize, standard and neck size packs)

1. Place cover on a flat surface with the pocket facing up and the flap open.
2. Remove pack from heating unit with tongs by gently lifting loop tab.
3. Place pack into cover; tuck flap into pocket, or close with hook and loop tabs (available on certain models). Note: neck covers do not have flaps.

Flat cover (oversize, standard and half size packs)

1. Place cover on a flat surface with the hook and loop facing up and the flap open.
2. Remove pack from heating unit with tongs by gently lifting loop tab.
3. Place pack on the left side of the cover
4. Fold cover over the pack. Make sure the hook/loop fasteners are aligned.
5. Press hook/loop up, pack in middle, 2nd cover hook/loop down).

Note: Oversize pack can be covered by 1 oversize cover or 2 standard covers (1st cover hook/loop up, pack in middle, 2nd cover hook/loop down).

Care for moist heat pack cover

Covers can be laundered in washing machine and dryer. Wash separately, no bleach. Max drying temperature: 160°F (-71°C). Fasten hook and loop closures together before laundering to avoid hook snagging terry material; some models include extra loop tabs to cover exposed hook during laundering. Avoid ironing.

Frequent use of moist heat pack

If pack is going to be used for regular or frequent treatments, keep it in water (either in the heating unit or in a covered pot on the stove).

Storage of moist heat pack

When pack is to be stored for an extended period of time, it should be kept moist and cold. Place wet pack in a plastic bag, seal it and store it in freezer. Alternatively, you can permit pack to dry thoroughly and then store it. When stored in this manner the filler becomes hard and caked. This does not harm the pack, but will require a much longer time for reheating.

CAUTION

- Do not apply hot pack directly to skin (always use towelling or hot pack covers)
- Individuals with circulatory problems should consult with physician before use
- Do not use over sensitive skin
- Do not sit or lie on pack
- Limit treatment to 30 minutes per application
- Do not use directly over cuts, abrasions or wounds
- Make sure pack is fully immersed in water before heating
- Do not add chlorine additive to water

terry covers

<i>Style (pack size)</i>	pack	<i>foam-filled</i>	<i>terry-filled</i>
standard - 10x12" / 25x30cm	11-1310	11-1360	11-1366
standard with pocket neck - 7x24" / 18x60cm	11-1311	11-1364 11-1361	11-1367
oversize - 15x24" / 38x61cm	11-1312	11-1362	11-1368
oversize with pocket half size (5x12")	11-1313	11-1365 11-1363	11-1369
circular - 10" / 25cm diameter	11-1205	11-1226	
hand - 7x13" / 13x30cm	11-1321		11-1376
dual hand			11-1377
spine small - 10x18" / 25x46cm	11-1314		
spine large - 10x24" / 25x61cm	11-1315		
knee-shoulder - 10x20"/25x51cm	11-1316		

half size



standard



oversize



circular



neck



Fabrication Enterprises Inc.
 250 Clearbrook Road, Suite 240
 Elmsford, NY 10523 (USA)
 tel: +1-914-345-9300 • 800-431-2830
 fax: +1-914-345-9800 • 800-634-5370
 FabEnt.com



Authorized CE representative:
 AJW Technology Consulting GmbH
 Königsallee 106
 40215 Düsseldorf (Germany)