


plastic 6-inch 360° Hi-Res[®] goniometer instructions for use

REF

12-1002HR

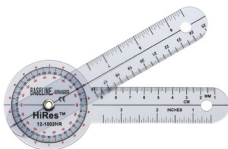

Intended Purpose: permits observation of joint's axis of motion, and range of motion

- 360° head has three scales for ease of use
- Primary scale reads in 1° increments
- Goniometer arm has a linear scale in inches and centimeters
- Goniometer arm remains at measurement until reset

other goniometer styles and sizes available:

12 inch 360°	12-1000*
8 inch 360°	12-1001*
6 inch 360°	12-1002*
6 inch 180° pocket.....	12-1005*
6 inch 360° rulongmeter	12-1006*
3 inch -30° to 120° finger.....	12-1012*

* available standard or with white HiRes[®] strip behind numbers (add HR to end of product code for HiRes[®] model)

Measurement Tolerance

This device measures inches and centimeters with a tolerance of ± 0.04 inches (± 1 millimeter) and degrees with a tolerance of ± 1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

⚠ WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Store flat to prevent warping
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



2862



AJW Technology Consulting GmbH
Breite Strasse 3
40213 Dusseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33, Suite 123
2595 AM, The Hague (NL)

©2025 FEI, all rights reserved.
Baseline and Hi-Res are
trademarks of Goldberg. rev0925