



**plastic 6-inch 180° Hi-Res®
pocket goniometer**

REF 12-1005HR



Intended Purpose

- permits observation of joint's axis of motion, and range of motion
- 180° head has three scales for ease of use
- primary scale reads in 1° increments
- goniometer arm has a linear scale in inches and centimeters
- goniometer arm remains at measurement until reset

other goniometer styles and sizes available:

12" 360°		12-1000*
8" 360°		12-1001*
6" 360°		12-1002*
6" 180° pocket		12-1005*
6" 360° rulongmeter	...	12-1006*
3" -30° to 120° finger	..	12-1012*

Measurement Tolerance
This device measures inches and centimeters with a tolerance of ±0.04 inches (±1 millimeter) and degrees with a tolerance of ±1 degree.

Disposal Method
Dispose of item in accordance with the local/ regional/national/international regulations.

WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Store flat to prevent warping
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device

FEL Fabrication Enterprises Inc.
265 Claverack Road, Suite 240
Huntington, NY 10821 (USA)
NY +1-514-345-9320 • 800-431-2830
FEL Inc. +1-514-345-9850 • 800-634-8370
FEL@fel.com

EC REP AJM Technology Consulting GmbH
Bielefelder Str. 9
40213 Düsseldorf Germany

Mediway NL-06-00000240
Postbus Margrietbloem 33, Suite 123
2555 AA, The Hague (NL)



©2025 FEL, all rights reserved.
Baseline and Hi-Res are
trademarks of Goldberg. rev0325