

plastic 6-inch 360° rulongmeter goniometer

REF

12-1006



Intended Purpose: permits observation of joint's axis of motion, and range of motion

- 360° head has three scales for ease of use
- primary scale reads in 1° increments
- goniometer arm has a linear scale in inches and centimeters
- goniometer arm remains at measurement until reset

other goniometer styles and sizes available:

12-inch 360°	12-1000*
8-inch 360°	12-1001*
6-inch 360°	12-1002*
6-inch 180° pocket.....	12-1005*
6-inch 360° rulongmeter	12-1006*
3-inch -30° to 120° finger....	12-1012*

* available standard or with white Hi-Res[®] strip behind numbers (add HR to end of product code for Hi-Res[®] model)

Measurement Tolerance

This device measures inches and centimeters with a tolerance of ± 0.04 inches (± 1 millimeter) and degrees with a tolerance of ± 1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

⚠ WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Store flat to prevent warping
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device



Fabrication Enterprises Inc
250 Clearbrook Road, Suite 240
Elmsford, NY 10523 (USA)
tel: +1-914-345-9300 • 800-431-2830
fax: +1-914-345-9800 • 800-634-5370
FabEnt.com



2862



AJW Technology Consulting GmbH
Breite Strasse 3
40213 Düsseldorf Germany



MedEnvoy
NL-IM-000000248
Prinses Margrietplantsoen 33, Suite 123
2595 AM, The Hague (NL)