
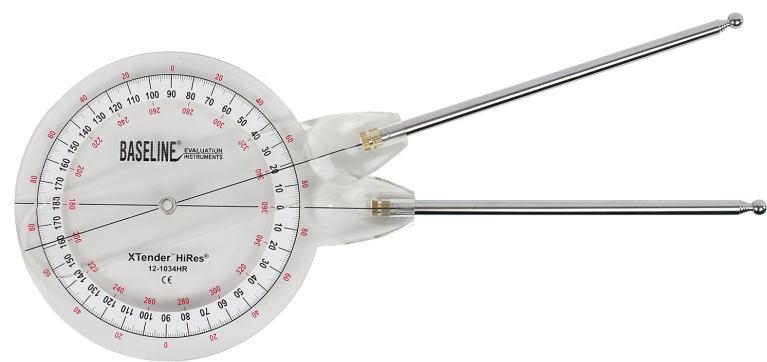


XTender[®] 360° Hi-Res[®]
extendable goniometer

REF

12-1034HR

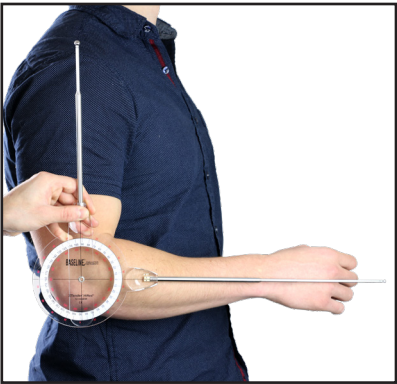

Intended Purpose: Permits observation of joint’s axis of motion and range of motion

- Measures large or small joints
- Stainless steel goniometer arms extend from 12.5" to 33"
- 360° head has three scales for ease of use (ISOM)
- Primary scale reads in 1° increments
- Goniometer arm remains at measurement until reset

Other Goniometer Styles and Sizes Available

12 inch 360°	12-1000*
8 inch 360°	12-1001*
6 inch 360°	12-1002*
6 inch 180° pocket.....	12-1005*
6 inch 360° rulongmeter	12-1006*
3 inch -30° to 120° finger.....	12-1012*

* available standard or with white Hi-Res[®] strip behind numbers
(add HR to end of product code for Hi-Res[®] model)

Measurement Tolerance

This device measures degrees with a tolerance of ±1 degree.

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

⚠

WARNING:

- Clean and disinfect device before and after each patient use
- Verify measurement markings are clearly visible
- Ensure smooth operation of all moving parts
- Align device properly with anatomical landmarks
- Apply gentle pressure during measurements
- Stop if patient reports pain or discomfort
- Replace if accuracy becomes compromised
- Do not attempt to bend or modify the device