

Collyer Breadth Caliper

User Manual

Intended Purpose

The Baseline[®] Collyer breadth caliper can be used to measure the width and offer accurate breadth readings for chest, shoulder, pelvis, etc. The 15-inch scale is marked in both inches and centimeters.

How to Use

1. Loosen the turn knob on caliper and place the desired body part (chest, shoulder, etc.) in between the caliper's jaws.
2. Bring the caliper jaws together until the object is firmly gripped. Then, tighten the caliper turn knob.
3. View your results on the scale, marked in inches and centimeters




Measurement Tolerance

This device measures with a tolerance of ± 0.04 inches (± 1 millimeter)

Disposal Method

Dispose of item in accordance with the local/regional/national/international regulations.

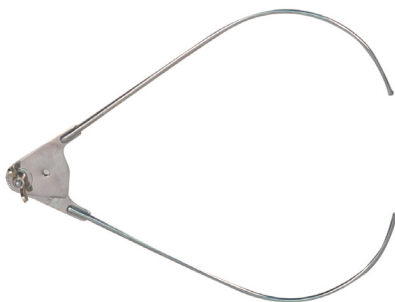

⚠ WARNING:

- Clean and disinfect the caliper before and after each patient use
- Verify device calibration before taking measurements
- Apply only gentle, consistent pressure when taking measurements
- Do not use on broken or irritated skin
- Stop measurement if patient reports discomfort
- Keep fingers clear of pinch points during operation
- Store in protective case when not in use
- Maintain calibration block properly