

Podiatry Whirlpools

Division of Acorn Engineering Company



A MEMBER OF





P Series 10 Gallon Mobile Whirlpool

P-10-M, P-10-MH, P-10-MU



P-10-MH

Whitehall P-10-M Mobile Whirlpools are recommended for treatment of the foot and ankle area, and when provided with a raised undercarriage, may also be used in treatment of the upper extremities.

Tank Capacity is 10 gallons.

P-10-MH is a Mobile Whirlpool with Handle of heavy duty construction with the hand grip section approximately 36" above ground level for ease in maneuvering the whirlpool from one location to another.

P-10-MU is a Mobile Whirlpool with Undercarriage of heavy duty construction and fixed height which raises the tank for use in treatment of the upper extremities.

Whirlpool Tank is 22″L x 13″W x 12″D and is seam less welded construction, fabricated from heavy gauge, type 304 stainless steel, polished to a satin finish. Tank rim is reinforced with a concealed stainless steel rod. Tank bottom is a seamless, coved (rounded) design for effective cleaning to minimize bacteria buildup.

Heavy Duty Swivel Casters support all mobile models. Rear pair of casters are lockable for stabilizing whirlpool during treatment.

Turbine Raising and Lowering Device permits adjustment of desired height and direction of water agitation towards the area of the body to be treated. It is spring balanced and provided with a locking device.

Turbine Assembly functions both as agitator and emptying device. It is equipped with Underwriters Laboratory listed 1/2 HP jet pump motor protected with an ABS plastic shell, automatic thermal overload protector, and lifetime-sealed bearings. All parts contacting the water are stainless steel, chrome plated brass or bronze.

Drain option -SDP has a bottom drain, leading to a separate, concealed water evacuation pump. A hose is attached to the side of the unit to facilitate the evacuation.

Thermometer is provided with a dual scale, adjustable calibration, and a stem inserts into a retainer to prevent rattle during whirlpool agitation.

GUIDE SPECIFICATIONS

Provide Whitehall Mobile Whirlpool (specify model number and options) with a 10-gallon capacity, 22"L x 13"W x 12"D. Whirlpool shall be fabricated from heavy gauge, type 304 stainless steel. Construction shall be seamless welded and exposed surfaces shall be polished to satin finish. Bottom of tank shall be a seamless, rounded coved design to minimize bacteria build up. Whirlpool shall have heavy duty swivel casters with locking device on rear pair of casters. Provide a turbine assembly with raising and lowering device which functions both as agitator and emptying device, permits adjustment of desired height, direction of water agitation, and can be locked into place. Turbine assembly shall be UL listed 1/2 HP jet pump motor with automatic thermal overload protector and lifetime-sealed bearings. Provide a thermometer with a stem retainer to prevent rattling when whirlpool is in operation.

Please visit www.whitehallmfg.com for most current specifications.





Whitehall Mfg. • P.O. Box 3527 • City of Industry, CA 91744 • (800) 782-7706 • (626) 968-6681 • Fax (626) 855-4862



Podiatry Whirlpools

MODEL NUMBER AND OPTION SELECTIONS Series ✓ P Podiatry Series **Capacity Type** ☐ -M Mobile Whirlpool Mobile Whirlpool with Handle ☐ -M-MH Mobile Whirlpool with Undercarriage ☐ -M-MU **Turbine Electrical Requirements** ☐ 115VAC / 50Hz ☐ 115VAC / 60HZ ☐ 230VAC / 50Hz ☐ 230VAC / 60HZ Country of Use

Refer to supplemental MXT15 sheet for details.

Suffix Options (See Whirlpool Options Section for Details) -ADH Auxiliary Drain Hose Assembly -AHC2 Adjustable High Chair with Rubber Tips ☐ -ALC Adjustable Low Chair □ -ALS Adjustable Low Stool □ -ARS Arm Rest Support ☐ -EG Enviro Glaze Color Powder Coating -Specify Color:_ (Special Finishes Consult Factory) ☐ -GFI Ground Fault Interrupter Hydro Lift (Refer To Page HL) -HL -PCS Pump Strainer Cover -SDP Separate Drain Pump

ELECTRIC REQUIREMENTS:
Amps: 6.9/6.2 Hertz: 50/60
Volts: 115 GFI Receptacle


For Mixing Valve:


ELECTRIC REQUIREMENTS:
Amps: 3.5/3.1 Hertz: 50/60
Volts: 230 GFI Receptacle

Certified per US & Canadian Medical Standards; UL 60601-1 & CSA C22.2 No. 601.1


Certified per European Standards; **CE** IEC 60601-1




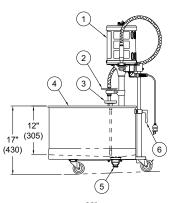

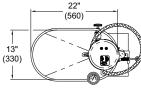

MODEL P-10-M

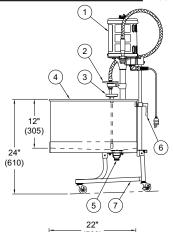

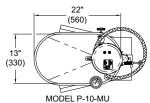

MOBILE P-10-M SERIES WHIRLPOOLS

NOTES:

- 1. 1/2 hp Jet Pump Motor
- 2. Variable Pressure Control
- 3. Thermometer
- 4. Stainless Steel Tank

- 5. Auxiliary Drain Valve
- 6. Turbine Raising & Lowering Device
- 7. Optional -MU Undercarriage

Please visit **www.whitehallmfg.com** for most current specifications.

Manufacturing, Member of Morris Group

International.

SELECTION SUMMARY & APPROVAL FOR MANUFACTURING	
Model Number & Options	Quantity
Company	Date
Contact	Title
Approval for Manufacturing/Signature	

Whitehall Mfg. • P.O. Box 3527 • City of Industry, CA 91744 • (800) 782-7706 • (626) 968-6681 • Fax (626) 855-4862