

lifetime limited warranty

Your Drive brand product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment. This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.

If you have a question about your Drive device or this warranty, please contact an authorized Drive dealer.



© 2012 Medical Depot, Inc. All rights reserved.

Drive is a trademark of Medical Depot, Inc.

Port Washington N.Y. 11050 USA Made in China

trapeze bar



www.drivemedical.com

REV3.1.6.14

assembly instructions

1. Slide bracket (14) over arm (21)
 - a. NOTE: Do not remove screw from arm. (20)
2. Set slide bracket to the desired position by tightening bolt (16).
3. Hook the chain (18) to the triangle (17).
4. Hook the unattached side of the chain to the slide bracket (14).
5. Install clamps to either bed headboard or trapeze base. Tighten firmly.
6. Slide trapeze arm (21) and trapeze post (19) over connector tube (24) and tighten with bolts (22).
7. Insert assembled, overhead trapeze (19 & 21) into receiving clamps.
8. Set height and tighten as necessary.

WARNING:

Prior to patient use, be sure to tighten trapeze clamps onto either bed or trapeze base.

The Trapeze, used in conjunction with a bed, can assist an individual weighing up to 250lbs (113kg). The Trapeze is designed to provide support, increase stability and assist the user when repositioning himself/herself in bed. The Trapeze is not designed, however, to support the total body weight of an individual. Use it for assistance only.

