


# SKILLBUILDERS®

## ADJUSTABLE SWING SEAT

### INSTRUCTIONS TO INSTALL CHAINS

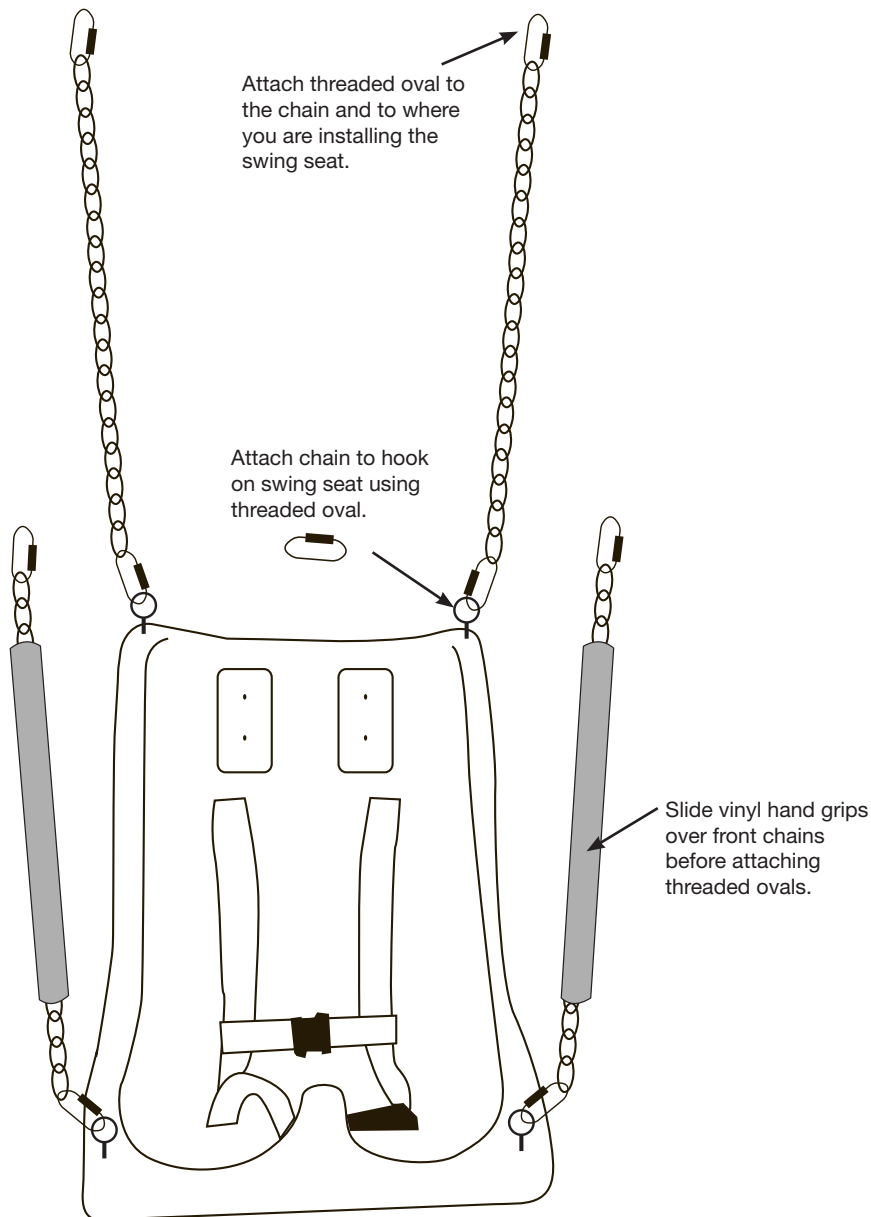

#### Swing seat with chain includes:

- (1) swing seat base
- (4) 72" (183 cm) chains
- (8) threaded ovals
- (2) vinyl hand grips

#### Set Up

1. Attach chains to swing seat base using threaded ovals
2. Slide vinyl hand grips over front chains
3. Attach threaded ovals to the other ends of the chains and connect to where you are installing the swing seat.

seat	inner dimensions	maximum weight capacity
small	20"L x 12"W x 9"D 51 cm x 30 cm x 23 cm	75 lb (34 kg)
medium	23"L x 15"W x 10"D 58 cm x 38 cm x 25 cm	150 lb (68 kg)
large	31"L x 20"W x 14"D 79 cm x 51 cm x 36 cm	200 lb (91 kg)
full-body reclining	24"L x 14"W x 11"D 61 cm x 36 cm x 28 cm	150 lb (68 kg)



**IMPORTANT SAFETY NOTICE:**  
To ensure proper seat tilt, make sure each side of chain is the same length.